



## Enzyme Cleaner

### - BRIEF DESCRIPTION -

*Enzyte Enzyme Cleaner is a multi-enzyme detergent concentrate formulated to be especially effective in the removal of proteins, blood, fatty tissues and other organic materials found on items such as instruments or on hard surfaces.*

*Enzyte Enzyme Cleaner is extremely effective when used as an ambient soak solution – no need to scrub! Because it is low foaming it can be used in ultrasonic cleaners, washers, and other automated equipment. Enzyte Enzyme Cleaner is non-toxic, non-caustic and 100 % biodegradable.*

### - USE PROCEDURE -

1. Shake bottle before each use.
2. Add one ounce of Enzyte Enzyme Cleaner per gallon of water. For automatic washers, please follow manufacturer's instructions.
3. Soak items to be cleaned until organic material is lifted off. Longer soaking time may be required for tough or dried on matter. When cleaning a surface completely wet the surface to be cleaned.
4. Flush the Enzyte Enzyme Cleaner through channels to ensure thorough cleaning.
5. Rinse items by aspirating water through all channels, hinges, etc. Completely rinse the Enzyte Enzyme Cleaner from items or surface.
6. Dry items before proceeding to the disinfection process.
7. Follow manufacturer's directions for sterilization.
8. Discard Enzyte Enzyme Cleaner after each use or when visibly contaminated.

### - SHELF LIFE -

There is no expiration for Enzyte Enzyme Cleaner. To ensure high activity levels and total rinsability, it is recommended to mix fresh solutions for each use.

### - PRECAUTIONARY STATEMENTS -

#### KEEP OUT OF REACH OF CHILDREN CAUTION

Use only as directed and read label and safety information before using. When handling concentrated enzyme preparations, as with most substances used in industrial processes, care should be taken to avoid skin contact and inhalation of aerosols. Personal protective equipment such as safety glasses and gloves is recommended.

### - PACKAGING -

1 Gallon/4 per case

### - CATALOG NUMBERS-

<u>Manufacturer</u>	<u>Item</u>	<u>Cat. No.</u>
Decon Labs, Inc.	1 Gallon	1901

For Additional Technical Information call  
Decon Customer Service  
(800) 332-6647, or  
See our website at [www.deconlabs.com](http://www.deconlabs.com)